




Precise Diagnostics for Improved Care


Software Download Instructions

If you are experiencing software download issues, it maybe because you have multiple copies on your desktop or hard drive. If you start over and downloaded the update again for any reason, please make sure to delete the first copy or you will create a copy with a version extension added. This version extension will not be read by the Vcheck and you will get an error. Below are the steps to solve this problem.


Please follow these steps. This is for Windows. (If you are using a Mac, your instructions may vary).

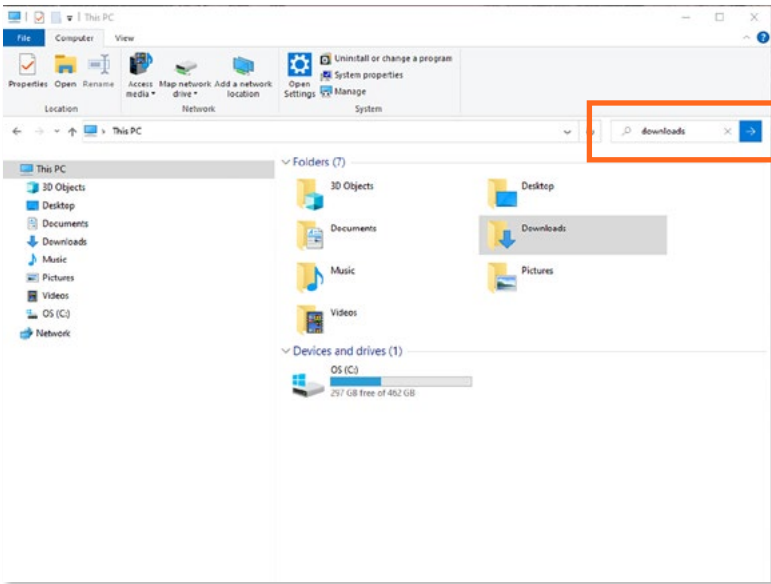


1. Make sure you check your download files. You can do this by typing "downloads" in your search bar.

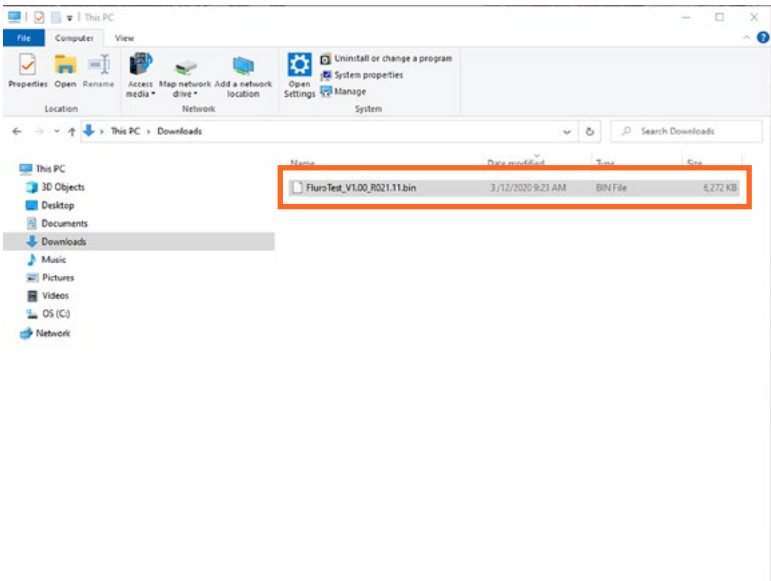


2. Delete any existing files containing "Fluro" or ".BIN"

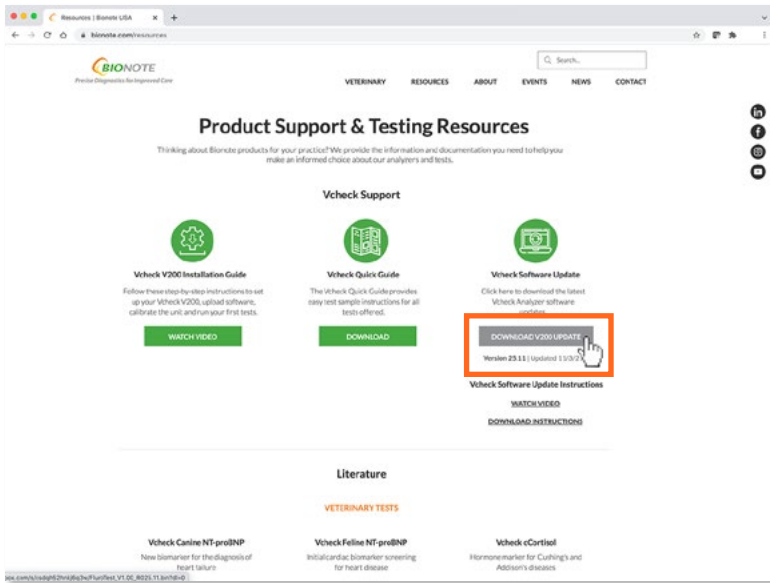


3. Go to our website by following this link: www.bionote.com/resources
Click on the "DOWNLOAD V200 UPDATE" button. (This will open a new tab/window.)

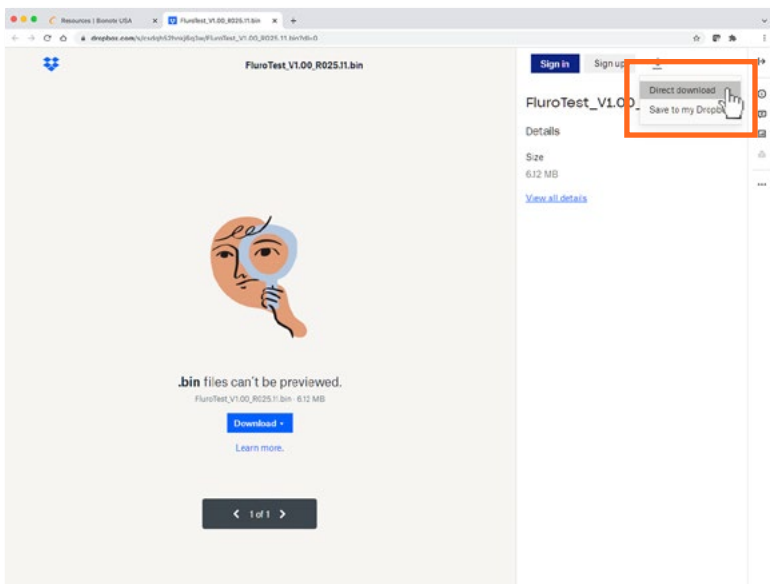

4. Click on the download arrow icon. Select "Direct download."
The file may automatically be downloaded to your computer's "Downloads" folder. Or you may be able to select the location of the file download.

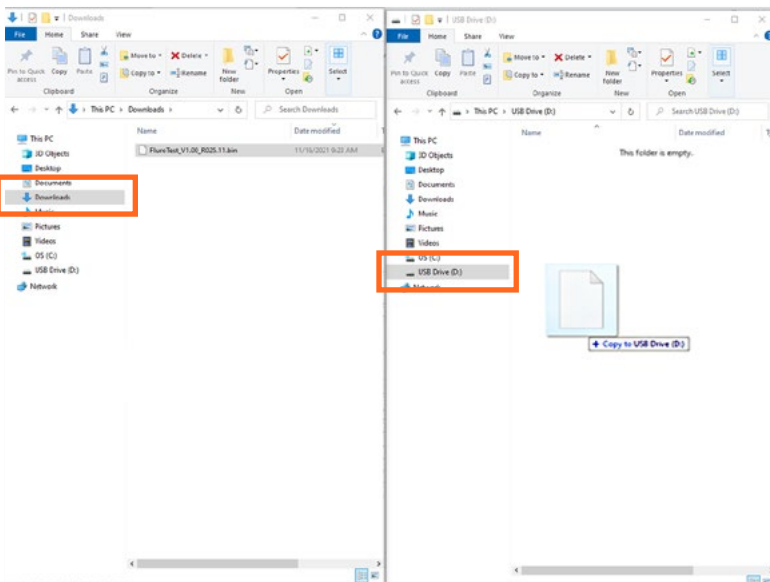

5. Insert a blank USB drive into your computer. Open a window of your "Downloads" folder (or the location of the file download you selected in the previous step). Open a window of the blank USB drive.

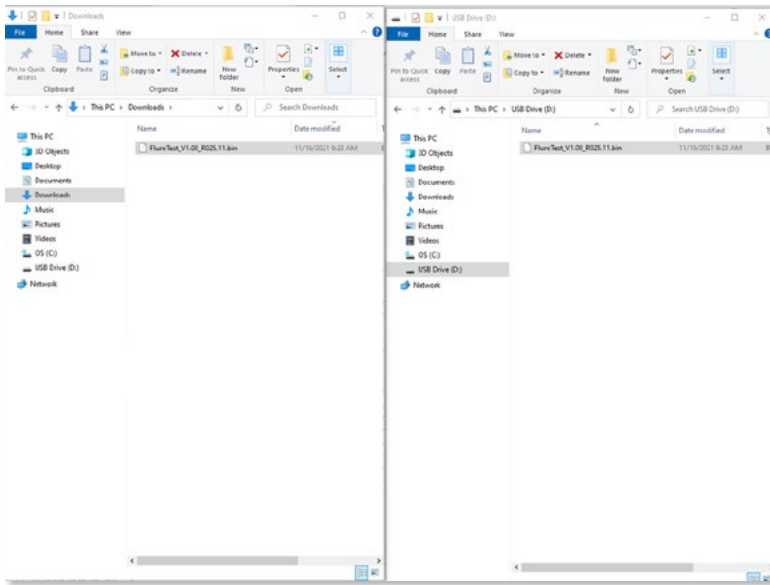

6. Safely eject the USB drive from your computer.
You are all set now! Simply update your analyzer per the directions on our website or as demonstrated during install.